Information au patient

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Patient Information

DONNING STEPLINE PLUS / STEPLINE HD

Caution : We dissuade from applying any type of lubricant to the stump or to STEPLINE PLUS / STEPLINE HD. Petroleum jelly will damage the liner and should be avoided. Your stump should be clean, dry, and free of soap residue.

1. Remove the pin before rolling the liner.
2. Turn STEPLINE PLUS / STEPLINE HD inside out so that the gel side is facing out.
3. With the logo and serial number on the front face of the liner, place the end of the liner against the end of your stump. If using a liner with distal attachment, centre the pin lock on the distal end of the stump.
4. Then carefully roll the liner onto your stump with the gel side against the skin. For optimal comfort, place the seams away from any sensitive area or bony prominence.

Caution : Please take care to roll the liner onto the stump instead of pulling it onto the stump because this could stretch the skin and result in discomfort or skin irritation.

5. Make sure that there are no wrinkles or trappings of air between your stump and the liner.
6. Don your prosthesis.

PERSPIRATION

It is not unusual for new STEPLINE PLUS / STEPLINE HD wearers to experience excessive perspiration during the first several weeks of the liner use. Some prosthetists have found the moderate application of unscented anti-perspirant or alum to the stump before donning the liner to be an effective method of controlling excessive sweating. Never use these products if there are open sores on the stump.

DAILY INSPECTION OF THE LINER

Each STEPLINE PLUS / STEPLINE HD must be inspected after each day of use. Unusual wear of the fabric or gel may indicate changes in the prosthesis fit. In order to preserve the warranty, be sure to contact your prosthetist immediately if any unusual wear is noticed. If you use a liner with distal attachment, check to ensure that the pin lock is secure.

CLEANING THE LINER

Caution : A proper hygiene is extremely important when using STEPLINE PLUS / STEPLINE HD. The liner must be thoroughly cleaned and dried at the end of each day of wear.

Daily cleaning

1. Remove the liner by unrolling it so that it comes off with the gel side facing out.
2. Immediately after removing the liner, with the gel side facing out, clean the liner thoroughly with hot tap water and mild soap. Apply the soap either by hand or with a clean, soft cloth or a sponge.

Caution : Do not scrub STEPLINE PLUS / STEPLINE HD. Scrubbing can roughen the surface of the liner, which can then cause irritations to the skin.

3. Take special care to rinse all soap residues from STEPLINE PLUS / STEPLINE HD
4. Dry the gel side of the liner with a clean, lint-free cloth.
5. Turn the liner inside out so that the fabric is on the outside and the gel is on the inside.
6. Place the liner, with the fabric side out, onto the drying stand.
7. Clean your stump.

Weekly cleaning (for disinfection)

1. Place a small amount of alcohol on a soft, clean cloth.
2. Lightly wipe the gel side of the liner with this cloth. Rinse with water to eliminate any excess of alcohol.
3. Place STEPLINE PLUS / STEPLINE HD, with the fabric side out, on the drying stand and allow it to dry.

Caution : Do not soak the liner in alcohol because large amounts of this product will stiffen it.

STORAGE OF STEPLINE PLUS / STEPLINE HD

When the liner is not being worn, it should be stored on the drying stand.

Caution :
It is extremely important to follow the above given instructions about donning, cleaning and care of your STEPLINE PLUS / STEPLINE HD. If not, this will void the warranty.

For further questions or information please contact :

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