STEPLINE[®] PLUS STEPLINE[®] HD

**Informations importantes destinées au Technicien Orthopédiste
Important information for orthopaedic technician
Wichtige Information für den Orthopädie-Techniker
Informaciones importantes para el técnico ortopédico**

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STEPLINE[®] PLUS

STEPLINE[®] HD

Important information for orthopaedic technician

**Please read this instruction leaflet thoroughly before using
STEPLINE PLUS / STEPLINE HD
with or without distal attachment.**

WARRANTY

PROTEOR HANDICAP TECHNOLOGIE offers a warranty of 180 days for any *STEPLINE PLUS / STEPLINE HD* with a defect in materials (replacement or credit note).

The warranty does not apply to the use of *STEPLINE PLUS / STEPLINE HD* with patients that do not correspond to the indications given in this document, or whose activity level is very high, or whose activities may cause an excessive use of *STEPLINE PLUS / STEPLINE HD* or may expose it to corrosive agents.

A failure to follow our given instructions will void the warranty.

RETURN GUIDELINES

Although *STEPLINE PLUS / STEPLINE HD* is accepted and used without problems by most patients, in some instances it might be necessary to return *STEPLINE PLUS / STEPLINE HD* to PROTEOR HANDICAP TECHNOLOGIE for replacement or credit.

Before returning *STEPLINE PLUS / STEPLINE HD* for any reason, please call PROTEOR HANDICAP TECHNOLOGIE. At this time, be ready to provide the serial number of the product and the reason for the return. A warranty replacement will be shipped upon your request.

You will be charged for the warranty replacement and a credit note will be issued upon determination that the returned *STEPLINE PLUS / STEPLINE HD* is indeed covered by warranty. If *STEPLINE PLUS / STEPLINE HD* is being returned for a reason other than a defect in materials and the sealed bag had been opened, then we will assume that the liner has been used and we will not make a replacement or issue a credit note.

If the warranty coverage on the return product has expired, or if it is determined that the damage to the product did not occur as a result of a defect in materials, then we will not issue any credit note.

Please check that *STEPLINE PLUS / STEPLINE HD* is returned in a state that is compatible with the safety of your employees and ours and if possible disinfect it.

MATERIAL SAFETY DATA SHEET

A material safety data sheet for *STEPLINE PLUS / STEPLINE HD* is available from us upon request.

1. INDICATIONS

STEPLINE PLUS / STEPLINE HD must mainly be used for stumps :

1. whose soft tissue is defective
>> elastomer acts as a substitute of the soft tissue
2. with important bony prominences
>> gel covers these pressure areas and increases comfort.
3. with burns (skin grafts)
>> when walking the liner reduces substantially the shear effect and protect the sensible skin areas against a too important mechanical load.
4. with skin deformities on the periosteum
>> most of the shear effect is reduced at the relevant skin areas.

There are more fitting possibilities with *STEPLINE PLUS / STEPLINE HD* than with a standard silicone liner and thanks to the elastomer fluidity it can be used with patients suffering from :

- problems of arterial circulation
- diabetes with skin sequels.

For these patients, it is important to check carefully the volume reduction and to take advantage of the possibilities of adhesion between *STEPLINE PLUS / STEPLINE HD* and the stump.

2. CONTRAINDICATIONS

1. Too much soft tissue or prominent soft tissue
>> the patient "swims" around the stump and is not safe when walking.
2. Unstable knee joint
>> Each short prosthesis causes knee joint instability.
A multi-axis soft foot can be interesting for these patients.
3. Very active patients
>> These patients suffer from a too high axis rise and a lack of rotation stability between the socket and the stump. Such a situation reduces substantially the liner resistance.
4. It is nearly impossible for the stump end to bear some weight
>> *STEPLINE PLUS / STEPLINE HD* cannot be used if the distal end of the stump cannot bear some weight. The stump end must generally bear from 20 % to 30 % of the patient body weight.
5. Poor hygiene
6. Patients with mental deficiencies
>> These patients do not understand the system enough.

3. PRELIMINARY ADVISE

Before proceeding to the adjustments of *STEPLINE PLUS / STEPLINE HD*, let the patient have *STEPLINE PLUS / STEPLINE HD* at home during at least a week to get used to handlings and new requirements concerning the stump skin. He will try *STEPLINE PLUS / STEPLINE HD* at rest, without his prosthesis in order to avoid shear effects that can, together with perspiration, cause water trappings.

Caution : To have a first donning of *STEPLINE PLUS / STEPLINE HD* that is comfortable for the patient, we recommend to use one of the two following methods : either preliminary stretching or customisation.

4. PRELIMINARY STRETCHING

Don *STEPLINE PLUS / STEPLINE HD* on a clean positive model that matches the stump dimensions and if possible is as long as *STEPLINE PLUS / STEPLINE HD*. Then remove it. This method is convenient for patients with small or medium stump. **If usual adjustment is not sufficient, proceed to customisation.**

5. CUSTOMISATION OF STEPLINE PLUS / STEPLINE HD

Recommended for patients with medium or large stump or important bony prominences. There are two possibilities : from a neutral plaster cast, make very carefully a smooth and clean positive model that will correspond thoroughly to the volume and the critical off-load areas. The unevenness of the positive model will appear in the gel at the same dimensions.

Method 1

1. Roll *STEPLINE PLUS / STEPLINE HD* onto a clean positive model of the stump.
2. Leave *STEPLINE PLUS / STEPLINE HD* on the positive model for at least one hour. The longer the liner is left on the positive model, the more the liner will take the shape of the model.
3. Remove *STEPLINE PLUS / STEPLINE HD* from the positive model.

Method 2

1. Roll *STEPLINE PLUS / STEPLINE HD* onto a clean and dry positive model of the stump. The positive model can be pre-heated at 65 °C during 10 mn.
2. Place the positive model, with the liner on it, into a pre-heated oven set at 65 °C.

Caution : Only the positive model must be in contact with the oven. A direct contact of *STEPLINE PLUS / STEPLINE HD* with any part of the oven may damage *STEPLINE PLUS / STEPLINE HD*. At the proximal end increase the length of the positive model of 10-15 cm in order to have a support area that will avoid to damage the gel.

3. Heat *STEPLINE PLUS / STEPLINE HD* in the oven for two to five minutes.
4. Remove the positive model with *STEPLINE PLUS / STEPLINE HD* from the oven. Allow *STEPLINE PLUS / STEPLINE HD* to cool approx. ten minutes before removing it from the model. Customisation can be vacuum made (about 20 % vacuum). In this case, remove the pre-heated plaster with *STEPLINE PLUS / STEPLINE HD* from the oven and suck with a PVA film.

6. DONNING OF STEPLINE PLUS / STEPLINE HD ON THE PATIENT

Caution : Do not apply any type of lubricant to the stump or to *STEPLINE PLUS / STEPLINE HD*. Petroleum jelly must be avoided because it will damage the liner. The stump should be clean, dry, and free of soap residue.

1. Turn *STEPLINE PLUS / STEPLINE HD* inside out so that the gel side is facing out.
2. Place *STEPLINE PLUS / STEPLINE HD* in order to have the logo and serial number in the front of the liner. Place the end of the liner against the end of the dry and clean stump. If using a liner with distal attachment, centre the pin lock on the distal end of the stump.
3. Then carefully roll *STEPLINE PLUS / STEPLINE HD* onto the stump with the gel side against the skin. For optimal comfort, place the seams away from any sensitive areas or bony prominences.

Caution : Please take care to roll the liner onto the stump instead of pulling it, which would stretch the skin and result in discomfort.

4. When donning *STEPLINE PLUS / STEPLINE HD* for the first time, it may feel slightly tight to the amputee. Use the following procedure for a comfortable fit: tuck the upper edge under, then roll the liner down the stump and stop rolling just prior to reaching the distal end. Then roll the liner up the stump for the second time. If the liner is still too tight, refer to the customization procedure.

5. Take care to avoid wrinkles or trappings of air between the stump and *STEPLINE PLUS / STEPLINE HD*.
6. If there is still space between *STEPLINE PLUS / STEPLINE HD* and the socket, use
 - a stump cap or
 - an additional soft socket.
7. The patient wears his socket. In case of a liner with distal attachment, follow the instruction leaflet of the specific locking device.

7. PERSPIRATION

It is not unusual for new *STEPLINE PLUS / STEPLINE HD* wearers to experience excessive perspiration during the first several weeks of the liner use. Some prosthetists recommend in this case the moderate application of unscented anti-perspirant or alum to the stump before donning the liner. Follow the instruction leaflet of these products and never use these products if there are open sores on the stump.

8. CUTTING OF STEPLINE PLUS / STEPLINE HD

For a better comfort and an easier donning on the patient, you can cut *STEPLINE PLUS / STEPLINE HD* to a shorter length if it is not fitted with distal attachment. For most of the patients you get the optimal length when you cut the liner between 8 cm and 15 cm above the brim of the socket. Do not cut *STEPLINE PLUS / STEPLINE HD* at the socket brim or at a lower level. For best results use a rotary cutter or big scissors.

Caution : A too long *STEPLINE PLUS / STEPLINE HD* is better than a too short one.

If you use a rotary cutter, place *STEPLINE PLUS / STEPLINE HD* on a cutting board or a thick piece of plastic. Use a straight edge as a guide and make the cutting wheel operate on the upper part of *STEPLINE PLUS / STEPLINE HD*. Check that the socket brim has no sharp edge or is covered with leather or another material because a sharp edge could make a hole in *STEPLINE PLUS / STEPLINE HD*.

Caution : If *STEPLINE PLUS / STEPLINE HD* is cut, then the drying/storage stand may also be cut if desired. Simply pull off the top plastic cap, cut the plastic tube to length and replace the cap.

9. SOCKET MANUFACTURE

If *STEPLINE PLUS / STEPLINE HD* is used in an already existing socket, the concept of total surface weight bearing must be strictly observed. The socket must not be damaged, rough or with a sharp edge. If necessary use a pad at the distal part of the stump to ensure a good contact in this area.

Caution : Using *STEPLINE PLUS / STEPLINE HD* in an already existing socket decreases substantially its resistance.

An optimal result is obtained only with a new manufactured socket. The following technique of manufacture of a new socket has been developed with experimented and qualified technicians and it is very efficient.

1. Don *STEPLINE PLUS / STEPLINE HD* as indicated in this document.
2. Cover *STEPLINE PLUS / STEPLINE HD* with a thin plastic film to keep the liner clean during the plaster casting process.
3. Don a thin Perlon sock over the plastic film.
4. Make the following marks on the Perlon sock :

- a) the highest, the lowest and the side-points of the patella
 - b) the normal fingerprints in either side of the patella tendon
 - c) the distal end and the cutting section of the tibia
 - d) the proximal, medial and lateral areas of the tibia
 - e) the head and the distal end of the fibula.
5. Measure the stump length with *STEPLINE PLUS / STEPLINE HD*.
 6. Cast plaster over *STEPLINE PLUS / STEPLINE HD* with minimal anatomical distortion. It is extremely important to capture the natural shape of the stump without moving the soft tissue. The two part plaster technique is efficient :
 - a) Use a non elastic bandage (to avoid distortion or moving of soft tissue). Cover the anterior face of the stump from the proximal and central area of the patella to the distal area of the tibia, as well as the round end of the tibia. Backwards the bandage must not go beyond the half of the most distal end. The medial and lateral edges of this plastered area must not go beyond the upper half of the stump. Smooth carefully the plaster with the hands and take care that you do not modify the stump shape. Then let the plaster set.
 - b) Cover the stump and the hardened plaster with plaster bandages in a spiral as usual. Take care that the soft tissue does not move.
 - c) Carefully smooth with the hands and shape the part of the cast corresponding to the patella tendon and the knee hollow. Make light thumbprints in either side of the patella tendon.
 7. Remove the cast and make a positive model. Notice the position of the patella, the distal end of the tibia, the medial and lateral parts of the tibia, as well as the head and the distal end of the fibula.
 8. Check that the model length is the same as the one measured in 5.
 9. Shape the plaster positive model in order to achieve a total surface weight bearing of the stump for optimal performance of *STEPLINE PLUS / STEPLINE HD*. To achieve this, the modifications indicated below can be made only on the plaster positive model. Additional layers on the model are neither recommended nor necessary.
 - a) reduce the patella tendon support by 10 to 15 mm
 - b) reduce by no more than 5 mm the area of the interosseal membrane
 - c) reduce by no more than 4 mm the area of the knee hollow
 - d) reduce by 4 % to 8 % the total plaster positive model. Reduction depends on tissue, irrigation and the patient activity level.
 - e) smooth the positive model.
 10. Manufacture a clear temporary socket with your usual method.
 11. Check precisely the stump position with *STEPLINE PLUS / STEPLINE HD* in the test socket.

Check that the stump has really full contact with the distal area of the socket whether it is bearing weight or not. There must not be any off-load area. Moreover, the patient must not feel uncomfortable in the distal end of the stump when the test socket bears total weight.

Caution : If the patient has still problems at the distal area, you can heat to make very light change in order to decrease distal pressure. However, in any case you must achieve full contact at the distal end of the socket.

12. As soon as the form of the test socket is satisfactory, you can let the patient bring it at home for trials during a week. Previously, secure it with a carbon fabric layer using the manual method.
13. After this trial period, a stump volume reduction is possible. Fill it up by using additional caps. When manufacturing the final socket, take care to reduce the plaster positive model according to the additional caps.
14. Manufacture the final socket with smooth contours and no sharp edges.

10. CLEANING OF STEPLINE PLUS / STEPLINE HD

Caution : A proper hygiene is extremely important when using STEPLINE PLUS / STEPLINE HD. Please review these care instructions very thoroughly with the amputee.

STEPLINE PLUS / STEPLINE HD must be thoroughly cleaned at the end of each day of wear and should dry for 24 hours, so that possible perspiration can get out of the material. As a consequence, each patient must have two *STEPLINE PLUS / STEPLINE HD*.

Daily cleaning

1. Remove the liner by unrolling it off, so that it comes off with the gel side facing out.
2. Immediately after removing *STEPLINE PLUS / STEPLINE HD*, with the gel side still facing out, clean *STEPLINE PLUS / STEPLINE HD* thoroughly with hot tap water (at a temperature that is comfortable for the hands) and a neutral PH soap. Apply the soap either by hand or with a clean, soft cloth or sponge.

Caution : Do not scrub STEPLINE PLUS / STEPLINE HD because it may roughen its surface, which may then cause irritations to the skin.

3. Take special care to rinse all soap residues from the liner.
4. Dry the gel side of the liner with a clean, lint-free cloth.
5. Turn *STEPLINE PLUS / STEPLINE HD* inside out so that the fabric is on the outside, and the gel is on the inside.
6. Place the liner, with the fabric on the outside, onto the supplied drying stand.
7. Clean the stump.

Weekly cleaning (for disinfection) :

1. Place a small amount of ethyl alcohol on a soft, clean cloth.
2. Lightly wipe the gel side of the liner with the cloth for two minutes.
3. Place *STEPLINE PLUS / STEPLINE HD*, with the fabric on the outside, on the drying stand and let it dry.

Caution : Do not soak STEPLINE PLUS / STEPLINE HD in ethyl alcohol because extended contact with large amounts of this product will stiffen it.

11. STORAGE OF STEPLINE PLUS / STEPLINE HD

When *STEPLINE PLUS / STEPLINE HD* is not being worn, it should be stored in a cool and dry place. If it is not worn for a long time, it must be stored on its storage stand in a clean plastic bag.

12. TROUBLE SHOOTING

PROBLEM	POSSIBLE CAUSE	SOLUTION
Fabric or gel breaking down in the anterior distal tibia area.	The socket is loose in that area, possibly due to a rise of the stump in the socket or a stump shrinkage.	Insert a pad in the socket in that area to tighten the socket fit.
Fabric wearing out along socket cut line.	Liner is rubbing against the cut line of the socket.	Smooth out the shape of the cut line. Make sure that the liner does not rub in the socket during flexion/extension. If necessary, add padding (such as leather) to the brim.
Excessive perspiration at new <i>STEPLINE PLUS / STEPLINE HD</i> users.	Body has not yet adjusted to <i>STEPLINE PLUS / STEPLINE HD</i> .	The perspiration should disappear after several weeks of <i>STEPLINE PLUS / STEPLINE HD</i> use. Some prosthetists have found that the moderate application of an unscented anti-perspirant or alum to the stump before donning the liner may be an effective method of controlling excessive perspiration. Always follow the instruction sheets of these products, and never use them if there are open sores on the stump.
Sudden appearance of sweating after a longer time of wearing <i>STEPLINE PLUS / STEPLINE HD</i> without sweating.	Air is getting between the liner and the stump, possibly due a stump atrophy.	Check the fit of <i>STEPLINE PLUS / STEPLINE HD</i> and the socket. Tighten the socket fit or choose another size for <i>STEPLINE PLUS / STEPLINE HD</i> .

PROBLEM	POSSIBLE CAUSE	SOLUTION
Skin irritation all over the stump.	STEPLINE PLUS / STEPLINE HD or the socket is loose, possibly due to a stump atrophy	Tighten the socket fit or choose another size for STEPLINE PLUS / STEPLINE HD to eliminate the looseness. To confirm that this is not an allergic reaction, perform a "patch test" : apply a piece of STEPLINE PLUS / STEPLINE HD material somewhere else on the body and look for a reaction.
Skin irritation	Poor hygiene.	Review proper cleaning, rinsing and disinfecting procedures with the patient.
	Use of an irritating lotion, cream or soap.	Check to see whether the patient has recently changed to a different cleaning product or has started using any lotion or cream containing hydrocarbon oils or animal fats or animal oils.
Skin irritation along the proximal edge of STEPLINE PLUS / STEPLINE HD.	Patient is pulling the liner up onto the skin instead of rolling it.	Refer to proper donning procedure of STEPLINE PLUS / STEPLINE HD.
Liner with distal attachment is pistoning up and down in the socket.	Distal end of the stump does not make contact with the distal end of the socket (stump may have shrunk distally).	Take care that the liner with distal attachment goes all the way to the distal end of the socket when the amputee is donning the prosthesis.
The stump and STEPLINE PLUS / STEPLINE HD rotate inside the socket.	Stump is extremely cylindrical, and the socket is loose.	Tighten the socket fit.
Gel is pulling away from the fabric along the proximal edge of STEPLINE PLUS / STEPLINE HD.	STEPLINE PLUS / STEPLINE HD is cut too short : the proximal edge of the liner is rubbing on the socket.	Change to a longer STEPLINE PLUS / STEPLINE HD.
Top edge of STEPLINE PLUS / STEPLINE HD is rolling down the leg.	STEPLINE PLUS / STEPLINE HD is too short.	Change to a longer STEPLINE PLUS / STEPLINE HD.
	STEPLINE PLUS is too tight at the proximal end.	Confirm that the patient is wearing the correct size of STEPLINE PLUS and make customisation if necessary.

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